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WARNING: This quick-user-guide does not replace the instructions manual. Start using SANY+AIR following details in point nr.1.

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ENG: The compressor is packed as shown in the picture.

Each box contains: SANY+AIR compressor, spiral hose with M/F connection, air-gun, instructions booklet and CE conformity certificate.

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Is it normal to have continuous air flow coming out from the air-gun nozzle?

Yes, this way the air-gun nozzle is kept clean.

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Is it normal that the liquid is sprayed only when pressing air-gun trigger?

Yes, the sanitizing liquid comes out only when needed.

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What kind of liquid can be used?

Any kind of sanitizing liquid can be used, whether concentrated or ready-to-use. Liquids with a high percentage of alcohol/spirit are very good disinfectants and they dry very quickly. We advise use of HACCP approved sanitizing products in case of contact with food or surfaces with direct contact with food. **WARNING!** take the safety precautions indicated on the container of the sanitizing product being used.

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La pistola e il tubo spiralato devono rimanere sempre collegati al compressore, prima dell'accensione fino a dopo lo spegnimento. I turned ON the compressor without connecting the air-gun. Now the compressor does not start and it is under strain. What can I do? The air-gun and the spiral hose must always be connected to the compressor, before starting it ON and once it is switched OFF.

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What is the size of the nebulizing drops?

The size of the drops can be regulated by a screw positioned on the air-gun, and it varies between 50 to 200 micron.

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How long does a bottle-full of sanitizing liquid last?

This depends on the nebulization dimensions, the time may vary from 4 to 20 minutes.

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Can I adjust the outgoing pressure of the compressor?

The compressor has been pre-set during its function test. The pressure regulator **MUST NOT BE TOUCHED**.

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Why does liquid leak from the container and the nozzle does not nebulize if I bend the air-gun?

The air-gun should not be bent or twisted over 45°. The container should be tightened well to the air-gun. **DO NOT OVERTURN** the air-gun.

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Does the SANY+AIR kit need maintenance?

The gasket of the air-gun liquid container should be replaced periodically, depending on the nr of hours of use.

Maintenance of the compressor should be done approximately every 1000 working hours.

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How large is the area covered by the nebulization spray?

The nebulization spray varies from 0,5mt to 1,5mt depending on the nebulization power set.

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How much sanitizing product can the liquid container hold?

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The air-gun trigger does not return to its position and nebulization does not stop. What shall I do?

The needle o-rings tend to dry out when using products with alcohol/spirit so you need to use an edible fat to lubricate the needle. Follow instructions below to perform this operation.